Istruzioni per l'uso
Instructions for use
Mode d'emploi
Bedienungsanleitung
Instrucciones para el uso

I

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GB

**RESUSCITATION KIT
OXYGEN TREATMENT UNIT
OXYGEN TREATMENT KIT**

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GB. Hereby the company TECNO-GAZ declares that the equipment reciprocal compatibility has been controlled according to the manufacturers instructions and then assembled following the instructions given by the manufacturer.

The complete kit has been packed according to information given by the manufacturer.

The internal of the kit undergoes adequate controls and checks that meet internal safety methods.

All is carried out to comply with the Art. 12 of the Directive 93/42/CEE.

Ref: Reanimation unit , Oxigenotherapy kit , Reanimation kit .

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WARRANTY

This equipment is covered by warranty for twelve months, with the exception of **moving membranes**, which are covered by warranty for six months.

This warranty starts on date of delivery of the equipment to the customer, as proved by the warranty coupon which has to be returned duly filled, stamped and signed by the seller.

In case of controversy, the date shown on the sales note will be applied.

Warranty repair or replacement of a part is at the manufacturer's unquestionable discretion and does not include personnel's transfer and package and transport freight.

Breakdowns resulting from poor maintenance, negligence, unskilful use or other causes not depending on the manufacturer will be excluded from warranty.

Wear parts and fittings are excluded from warranty.

The manufacturer does not acknowledge the right of replacement of the complete equipment.

This warranty does not imply any compensation for direct or indirect damages of any kind to people or property due to equipment inefficiency.

This warranty will automatically void if the equipment is tampered with, repaired or altered by the buyer or any unauthorised third party.

When servicing is required, the buyer shall only apply to the seller or services indicated by the manufacturer.

Components replaced under warranty shall be returned to TECNO-GAZ free of carriage.

If components are not returned, their cost will be charged to the applicant.

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| INDEX | | |
|--------------|--|-------------|
| PAR | DESCRIPTION | PAGE |
| 01 | INFORMATION TO THE USER | 1-6 |
| 02 | PRODUCT'S INTRODUCTION | 16-17 |
| 03 | TECHNICAL FEATURES | 17 |
| 04 | PACKING - TRANSPORT - STORAGE - INSTALLATION - DEMOLITION - DISPOSAL | 17 |
| 05 | WORKING AND USE | 18 |
| 06 | INSTRUCTIONS FOR USER | 18-21 |
| 07 | ORDINARY AND EXTRAORDINARY MAINTENANCE | 22 |
| 08 | ANALYSIS AND RESEARCH OF FAILURES | 22 |
| 09 | COMMERCIAL FEATURES, SPARE PARTS AND DOCUMENTATION | 22 |
| 10 | DIAGRAMS | 22 |

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Medical Devices List

| ITEM | DESCRIPTION |
|------------|--|
| 1910-B | Oxygen treatment kit in case with handle (basic size: 110x170 height: 430 weight 3.1Kg containing bottle 0.480 litres Italian inlet, reducer R400 with distributor, mask for oxygen treatment with measuring device). |
| 1910-B-D | Oxygen treatment kit in case with handle (basic size: 110x170 height: 430 weight 3.1Kg containing bottle 0.480 litres German inlet (G 3/4" DIN 477 Nr. 9), reducer R400 with distributor, mask for oxygen treatment with measuring device). |
| 1910-B-F | Oxygen treatment kit in case with handle (basic size: 110x170 height: 430 weight 3.1Kg containing bottle 0.480 litres French inlet (Ø 22,91x1,814 NF E29-650/F), reducer R400 with distributor, mask for oxygen treatment with measuring device). |
| 1911-B | Oxygen treatment kit in case with handle (basic size: 110x170 height: 430 weight 4.5Kg containing bottle 2 litres Italian inlet, reducer R400 with distributor, mask for oxygen treatment with measuring device). |
| 1911-B-D | Oxygen treatment kit in case with handle (basic size: 110x170 height: 430 weight 4.5Kg containing bottle 2 litres German inlet (G 3/4" DIN 477 Nr. 9), reducer R400 with distributor, mask for oxygen treatment with measuring device). |
| 1911-B-F | Oxygen treatment kit in case with handle (basic size: 110x170 height: 430 weight 4.5Kg containing bottle 2 litres French inlet (Ø 22,91x1,814 NF E29-650/F), reducer R400 with distributor, mask for oxygen treatment with measuring device). |
| 1912-B | Oxygen treatment kit in case with handle (basic size: 165x165 height : 600 weight 10Kg containing bottle 5 litres Italian inlet, reducer R400 with distributor, mask for oxygen treatment with measuring device). |
| 1912-B-D | Oxygen treatment kit in case with handle (basic size: 165x165 height: 600 weight 10Kg containing bottle 5 litres German inlet (G 3/4" DIN 477 Nr. 9), reducer R400 with distributor, mask for oxygen treatment with measuring device). |
| 1912-B-F | Oxygen treatment kit in case with handle (basic size: 165x165 height: 600 weight 10Kg containing bottle 5 litres French inlet (Ø 22,91x1,814 NF E29-650/F), reducer R400 with distributor, mask for oxygen treatment with measuring device). |
| 1916-B | Oxygen treatment kit Mod. "OXYKIT" (size 510x400x130 weight 4.5Kg) (containing 2 oxygen bottle 0.480 litres Italian inlet, 1 reducer R400 with distributor, mask for oxygen treatment with measuring device). |
| 1916-B-D | Oxygen treatment kit Mod. "OXYKIT" (size 510x400x130 weight 4.5Kg) (containing 2 oxygen bottle 0.480 litres deutsch inlet, 1 reducer R400 with distributor, mask for oxygen treatment with measuring device). |
| 1916-B-F | Oxygen treatment kit Mod. "OXYKIT" (size 510x400x130 weight 4.5Kg) (containing 2 oxygen bottle 0.480 litres french inlet, 1 reducer R400 with distributor, mask for oxygen treatment with measuring device). |
| 3003-E | Oxygen treatment unit on trolley mod. AIR-OX (package size 35.7x19.7x72H weight 14.5Kg containing bottle 5 litres Italian inlet, reducer R400 with flowmeter and washer, bag for adults, face mask size 4, helic mouth opener, tongue puller, Guedel cannula sizes 1-2, ox. tube L=200cm). |
| 3003-E-D | Oxygen treatment unit on trolley mod. AIR-OX (package size 35.7x19.7x72H weight 14.5Kg containing bottle 5 litres German inlet (G 3/4" DIN 477 Nr. 9), reducer R400 with flowmeter and washer, bag for adults, face mask size 4, helic mouth opener, tongue puller, Guedel cannula sizes 1-2, ox. tube L=200cm). |
| 3003-E-F | Oxygen treatment unit on trolley mod. AIR-OX (package size 35.7x19.7x72H weight 14.5Kg containing bottle 5 litres French inlet (Ø 22,91x1,814 NF E29-650/F), reducer R400 with flowmeter and washer, bag for adults, face mask size 4, helic mouth opener, tongue puller, Guedel cannula sizes 1-2, ox. tube L=200cm). |
| 3007-E-2 | Resuscitation Kit in case Mod. SPEEDY 2 (size 510x400x130H weight 4.5Kg containing: bottle 0.480 litres Italian inlet, reducer R400 with distributor, bag for adults, spare mask for adults, helic mouth opener, tongue puller, Guedel cannula sizes 1-2, ox. tube L=200cm). |
| 3007-E-2-D | Resuscitation Kit in case Mod. SPEEDY 2 (size 510x400x130H weight 4.5Kg containing: bottle 0.480 litres German inlet (G 3/4" DIN 477 Nr. 9), reducer R400 with distributor, bag for adults, spare mask for adults, helic mouth opener, tongue puller, Guedel cannula sizes 1-2, ox. tube L=200cm). |
| 3007-E-2-F | Resuscitation Kit in case Mod. SPEEDY 2 (size 510x400x130H weight 4.5Kg containing: bottle 0.480 litres French inlet (Ø 22,91x1,814 NF E29-650/F), reducer R400 with distributor, bag for adults, spare mask for adults, helic mouth opener, tongue puller, Guedel cannula sizes 1-2, ox. tube L=200cm). |
| 3007-R-3 | Resuscitation Kit in case Mod. SPEEDY 2 (size 510x400x130H weight 2.2Kg containing: bag for adults, spare mask for adults, helic mouth opener, tongue puller, Guedel cannula sizes 1-2-3, ox. tube L=200cm). |
| 3007-R-6 | Resuscitation Kit in case Mod. SPEEDY 2 (size 510x400x130H weight 2.2Kg containing: bag for children, autoclavable face mask size 1, helic mouth opener, tongue puller, Guedel cannula sizes 1, 0-1-2, ox. tube L=200cm). |
| 4500-E | Resuscitation Kit in case Mod. OXYSET (size 385x240x135H weight 4,2Kg containing: bottle 0.480 litres Italian inlet, reducer R400 with distributor, bag for adults, face mask size 4, helic mouth opener, tongue puller, Guedel cannula sizes 1-2-3, ox. tube L=200cm). |

| | |
|----------|--|
| 4500-E-D | Resuscitation Kit in case Mod. OXYSET (size 385x240x135H weight 4,2Kg containing bottle 0.480 litres German inlet (G 3/4" DIN 477 Nr. 9), reducer R400 with distributor, bag for adults, face mask size 4, helic mouth opener, tongue puller, Guedel cannula sizes 1-2-3, ox. tube L=200cm). |
| 4500-E-F | Resuscitation Kit in case Mod. OXYSET (size 385x240x135H weight 4,2Kg containing bottle 0.480 litres French inlet (Ø 22,91x1,814 NF E29-650/F), reducer R400 with distributor, bag for adults, face mask size 4, helic mouth opener, tongue puller, Guedel cannula sizes 1-2-3, ox. tube L=200cm). |
| 5000-E | Resuscitation Kit in case Mod. OXYCASE (size 420x330x150H weight 5.5Kg) containing bottle 0.480 litres Italian inlet , reducer R400 with distributor, bag for adults, face mask sizes 3-4-5, helic mouth opener, tongue puller, Guedel cannula sizes 1-2-3, ox. tube L=200cm). |
| 5000-E-D | Resuscitation Kit in case Mod. OXYCASE (size 420x330x150H weight 5.5Kg) containing bottle 0.480 litres German inlet (G 3/4" DIN 477 Nr. 9), reducer R400 with distributor, bag for adults, face mask sizes 3-4-5, helic mouth opener, tongue puller, Guedel cannula sizes 1-2-3, ox. tube L=200cm). |
| 5000-E-F | Resuscitation Kit in case Mod. OXYCASE (size 420x330x150H weight 5.5Kg) containing bottle 0.480 litres French inlet (Ø 22,91x1,814 NF E29-650/F), reducer R400 with distributor, bag for adults, face mask sizes 3-4-5, helic mouth opener, tongue puller, Guedel cannula sizes 1-2-3, ox. tube L=200cm). |
| 6000-E | Resuscitation Kit in case Mod. OXYBAG 1 (size 360x250x150H weight 3.2Kg, containing bottle 0.480 litres Italian inlet , canvas bag, reducer R400 with distributor, bag for adults, face mask sizes 4, helic mouth opener, tongue puller, Guedel cannula sizes 1-2-3, ox. tube L=200cm). |
| 6000-E-D | Resuscitation Kit in case Mod. OXYBAG 1 (size 360x250x150H weight 3.2Kg, containing bottle 0.480 litres German inlet (G 3/4" DIN 477 Nr. 9), canvas bag, reducer R400 with distributor, bag for adults, face mask sizes 4, helic mouth opener, tongue puller, Guedel cannula sizes 1-2-3, ox. tube L=200cm). |
| 6000-E-F | Resuscitation Kit in case Mod. OXYBAG 1 (size 360x250x150H weight 3.2Kg, containing bottle 0.480 litres French inlet (Ø 22,91x1,814 NF E29-650/F), canvas bag, reducer R400 with distributor, bag for adults, face mask sizes 4, helic mouth opener, tongue puller, Guedel cannula sizes 1-2-3, ox. tube L=200cm). |
| 6000/R-2 | Resuscitator Mod. OXYBAG 2 (size 360x250x150H weight 2.2Kg containing canvas bag, bag for adults, face mask size 4, helic mouth opener, tongue puller, Guedel cannula sizes 1-2-3, ox. tube L=200cm). |

PAR 03 TECHNICAL FEATURES

The technical features of each equipment are described in paragraph 02 "Product presentation"

PAR 04 PACKING - TRANSPORT - STORAGE - INSTALLATION - DEMOLITION - DISPOSAL

04.1 - TRANSPORT - STORAGE- INSTALLATION – SCRAPPING

Transport and storage must take place in keeping with what is described on the label which is stick on the package

04.2 INSTALLATION

All the devices are ready-made and do not need any installation

04.3 – SCRAPPING

The disposal of the package components and of the device container must be carried out according to the laws of the destination country.

Materials: carton, cellophane, polyurethane, polypropylene, iron, canvas

Directions for scrapping and demolition are included in the instructions for use of each medical equipment.

The disposal of the ox. bottles must be carried out according to the laws of the destination country.

It is compulsory to empty the bottle carefully and to leave the valve open. ATTENTION! In any case keep a copy of the certificate which is issued by the bottle manufacturer

PAR 05 WORKING AND USE

See par. 06 for working (“Instructions and warnings”).

-  The use of the equipment is reserved to authorised or trained personnel. The improper use or negligence on the part of the user can cause serious damages to persons and goods. TECNO-GAZ is not to be held responsible for them.

PAR 06 INSTRUCTIONS FOR USER

WARNINGS

-  Take out of the package paying attention not to damage the components
-  Make sure that the contents do not bear transport damages and that they correspond to what you have requested.
-  The work position of the bottles with flowmeter and washer must be vertical
-  Read carefully the reducer R400 handbook
-  Place the equipment away from sources of heat and electric machines.
-  The oxygen supply inside the resuscitation kit and the kit for oxygen treatment takes place with a constant flow (which is not adjustable) at 10 NI/min.
-  The bottle does not contain any oxygen; for filling apply to the medical gas refilling point (always bring the bottle certificate with you)
-  THE APPLIANCE HAS NOT TO BE USED IN POTENTIAL EXPLOSIVE ATMOSPHERE.

INSTRUCTIONS FOR USE

RESUSCITATION UNIT

- 1 Connect the mask **(A)** to the bag's outlet valve **(B)** (See fig. 1).
- 2 Insert the end **(C)** of the tube **(D)** in the inlet **(E)** which is inside the bag's underneath knob **(F)** (See Fig. 1).
- 9 Fill the bottle **(Z)** with medical oxygen
- 10 Screw the reducer **(B)** on the bottle **(Z)** by hand and pay attention that you place the flowmeter in vertical position (read carefully the reducer R400 handbook before performing this operation)
- 11 Screw the washer **(G)** under the flowmeter **(L)**
- 12 Join the other end of the tube **(D)** to the washer's outlet **(G)**. (See Fig. 2).
- 13 Fill the washer **(G)** with demineralised water in the desired quantity (See Fig. 2/a).
- 14 Let the oxygen flow by turning very slowly the tap **(H)** in the direction indicated on the tap itself “OPEN”, measure out the flow quantity by means of the flowmeter dial **(I)** checking the reading on the small column **(L)**; this operation will allow a richer oxygen ventilation (Fig. 2).

FIG.1

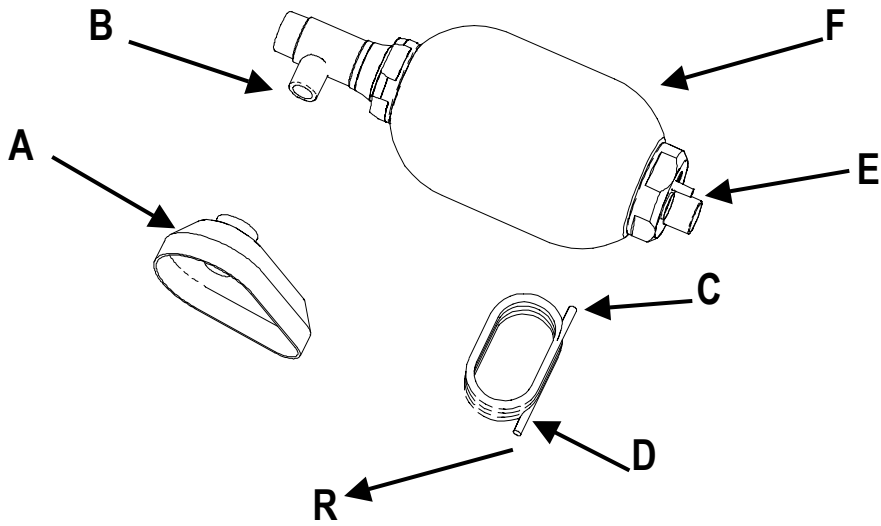
FIG.2


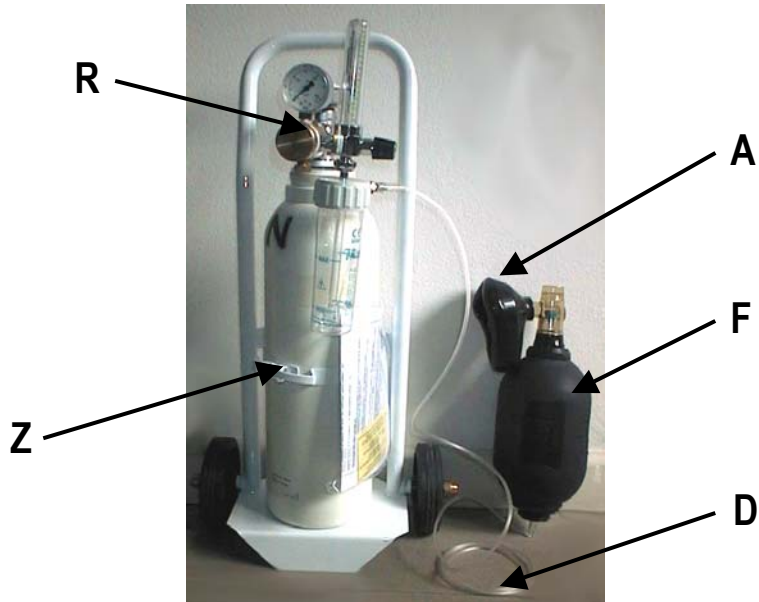
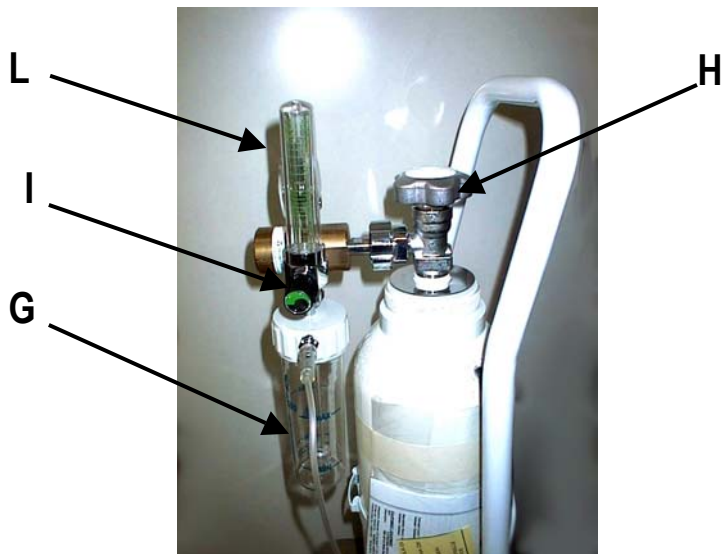
FIG.2/A

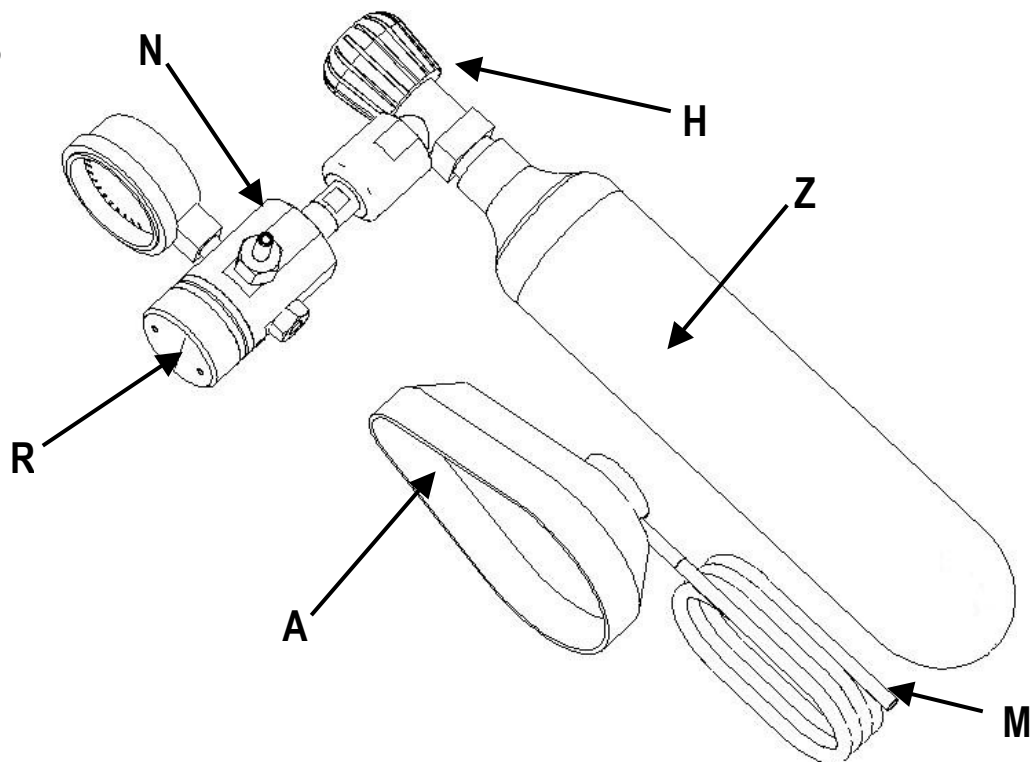
OXYGEN TREATMENT KIT

- 7) Fill the bottle (Z) with medical oxygen
- 8) Screw the reducer (R) on the bottle by hand and pay attention that you place the fluxmeter in vertical position (read carefully the reducer R400 handbook before performing this operation)
- 9) Connect the end (M) of the mask's (A) link tube to the outlet (N) of the reducer R400 SRPB404 (See Fig. 3).
- 10) Place the mask (A) on the patient's face following the mask's instructions (Fig. 3).
- 11) Let the oxygen flow by turning very slowly anticlockwise the tap (H) of the bottle (Fig. 3)
- 12) Treat the patient respecting the due rules.

FIG.3

RESUSCITATOR KIT

- 8 Fill the bottle with medical oxygen
- 9 Screw the reducer on the bottle by hand and pay attention that you place the manometer in vertical position (read carefully the reducer R400 handbook before performing this operation)
- 10 Connect the mask (A) to the outlet of the bag' valve (F).
- 11 Join the end (C) of the tube (D) to the proper inlet (E) which is on the bag's underneath knob.
- 12 Insert the end (T) of the tube in the outlet (N) of the reducer (R) which is mounted on the bottle (Z).
- 13 Let the oxygen flow by turning very slowly anticlockwise the tap F of the bottle (see Fig. 4).
- 14 Treat the patient respecting the due rules.

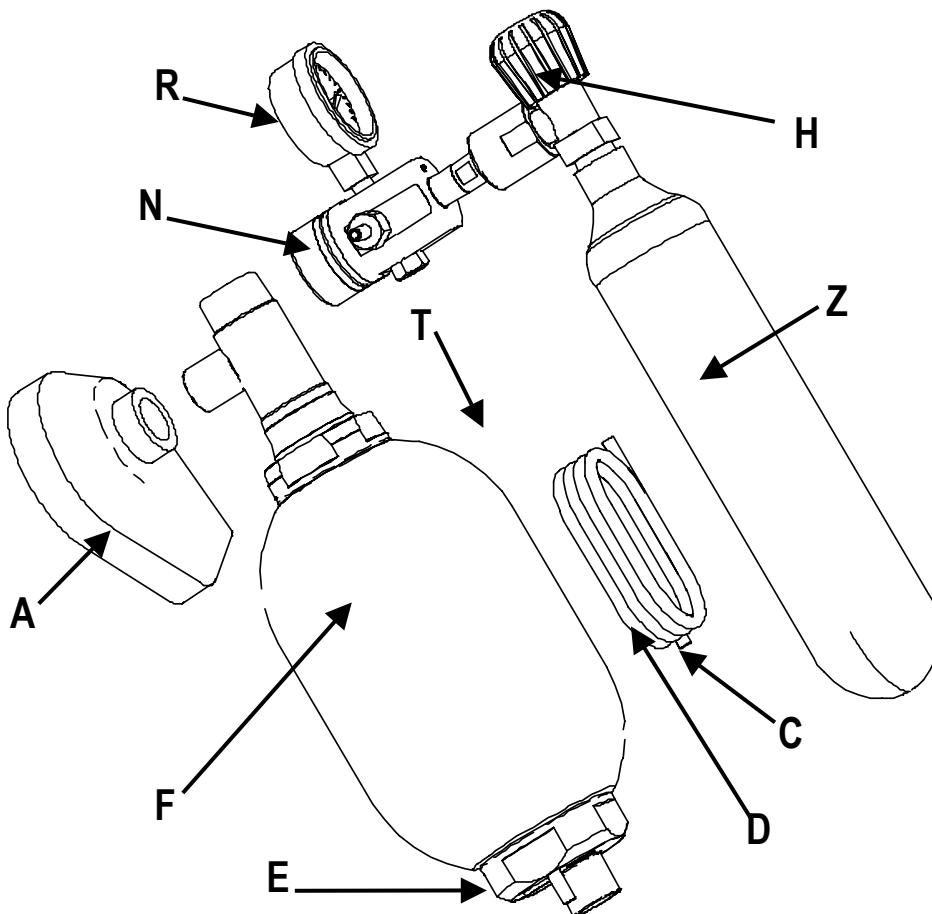FIG.4

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As for the maintenance refer to the instruction manual of each device

 Read carefully the above-mentioned manuals before use or maintenance.

PAR 08 ANALYSIS AND RESEARCH OF FAILURES

In case of working defects of the devices refer to the relative manuals.

PAR 09 COMMERCIAL FEATURES, SPARE PARTS AND DOCUMENTATION

TECNO-GAZ does not supply spare parts for the single devices but only the complete equipment.

PAR 10 DIAGRAMS

See the "Instructions for use" and the figures.

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TECNO-GAZ S.p.A.