TECNOPRINT

ART. 2110S

<i>Istruzioni per l'uso</i>	ITALIANO
<i>Instructions for use</i>	ENGLISH
<i>Instructions d'utilisation</i>	FRANÇAIS
<i>Bedienungsanleitung</i>	DEUTCH

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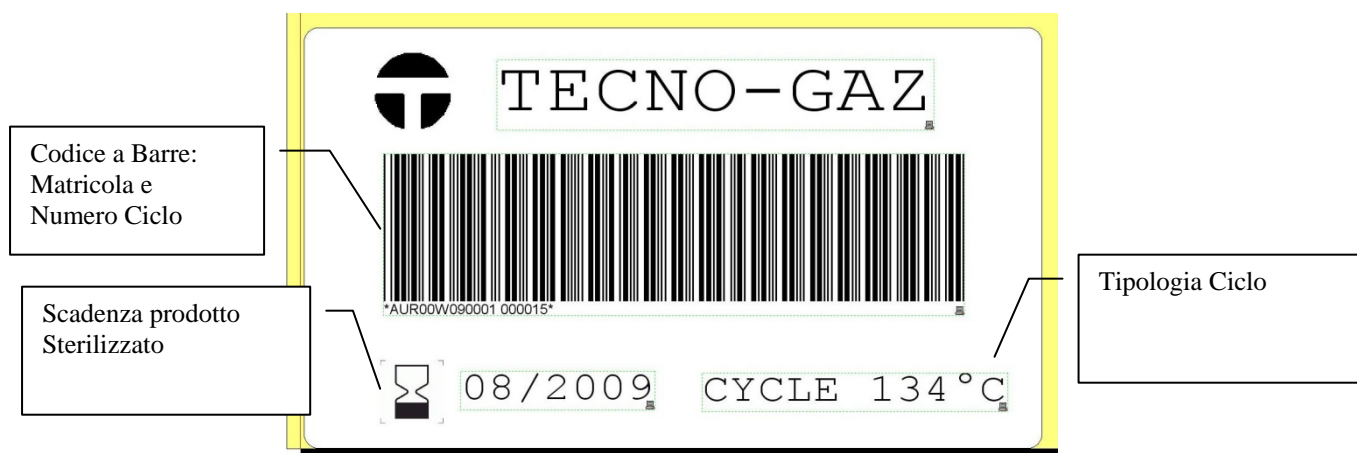
FORMATO ETICHETTA TRACCIABILITA'

INSTALLATION INSTRUCTIONS

Open the package and get out the items.
Look at the printer's manual to setup the ink's ribbon and labels.

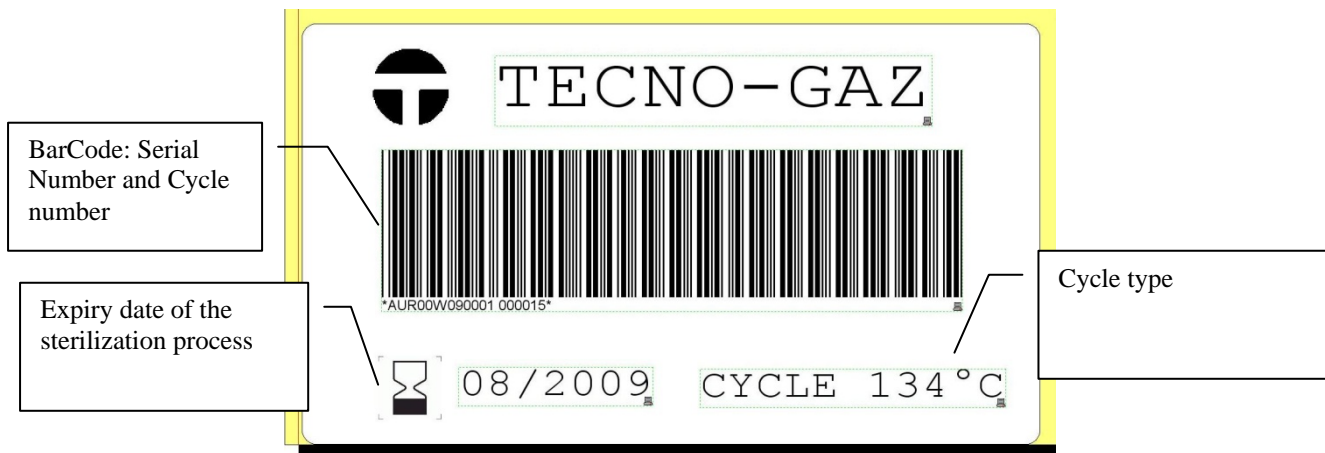
Plug-in the Male connector of the RS-232 cable into the printer and the female connector to the autoclave.
Plug-in the power cable and power on the printer.
Push three time the "feed" button to let the optical labels sensor to align.

Power on the autoclave and push the "Setup" button.
Select the item "External Printer" and enter into the sub-menu.
Set the "Kind" field to "Traceability".

Then select "Print Test" and the autoclave will start printing a predefined label.
If the print is right then the setup procedure has been completed.

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The diagram shows a label with the following elements:

- Logo:** A stylized 'T' inside a circle.
- Brand Name:** TECNO-GAZ
- Barcode:** A standard 1D barcode with the alphanumeric string *AUR00W090001 000015* below it.
- Expiration Date:** 08/2009, accompanied by an hourglass icon.
- Cycle Information:** CYCLE 134 °C.

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EN

This manual must always be kept with the product, in complying with the Directives of European Community. TECNO-GAZ reserves the right to modify the enclosed document without notice. TECNO-GAZ reserves the property of the document and forbids others to use it or spread it without its approval.

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TECNO-GAZ S.p.A.