Molift Air Tilt

molift[®]
by Etac

User manual - English

Bruksanvisning - Svenska

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


BM14099 Rev. E 2021-12-23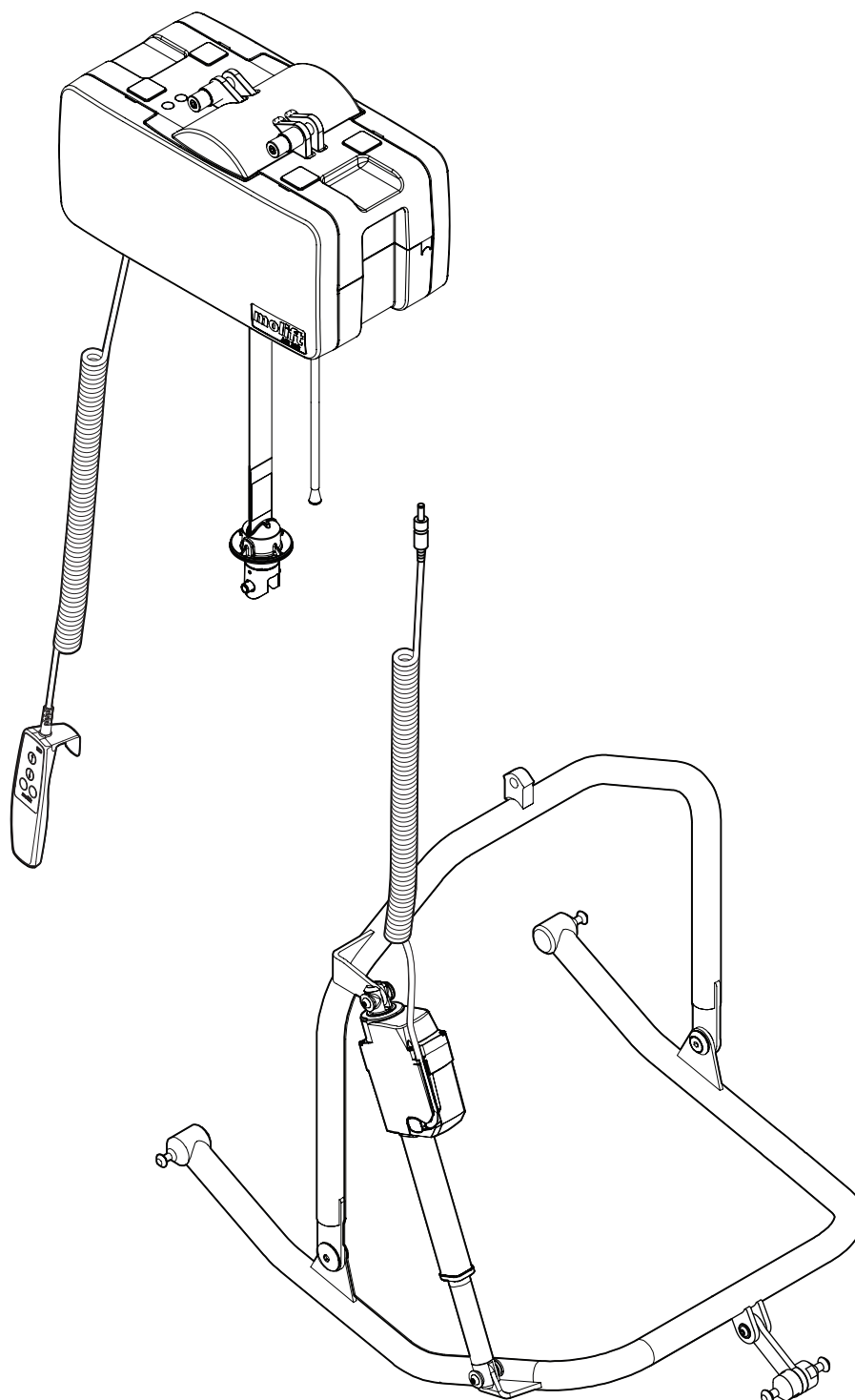

MD
CE

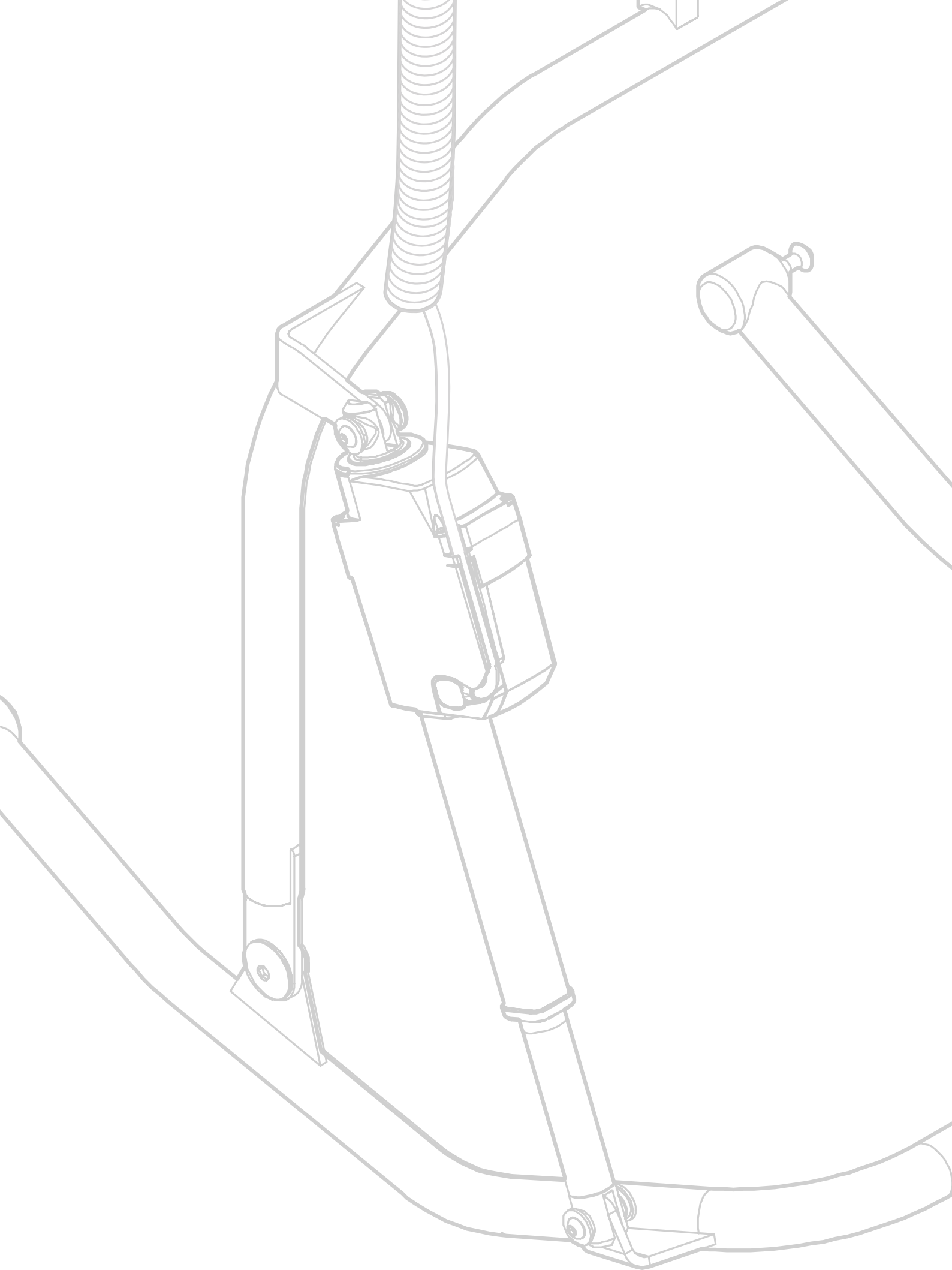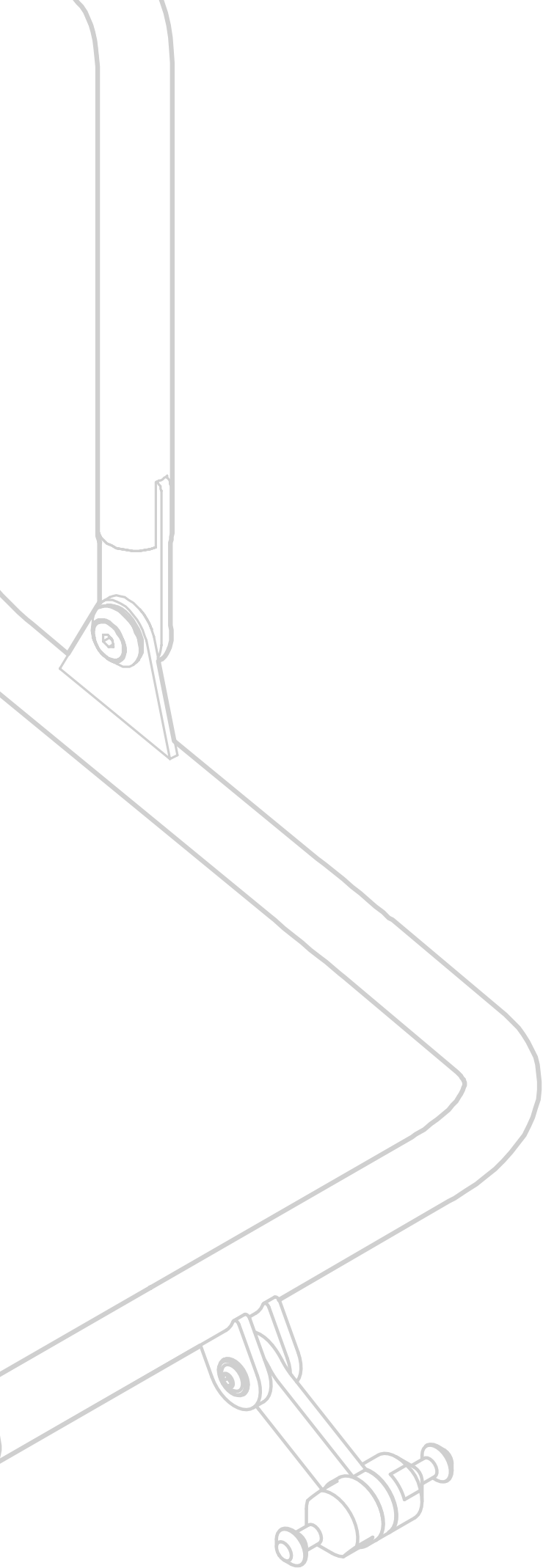Contents

User manual - English - 1
Bruksanvisning - Svenska - 7
Brukermanual - Norsk - 13
Brugsvejledning - Dansk - 19
Käyttöohje - Suomi - 25
Gebrauchsanweisung - Deutsch - 31
Handleiding - Nederlands - 37
Manual d`utilisation - Francais - 43
Manual utente - Italiano - 49
Manual de usuario - Español - 55

Important

This User Manual contains important safety instructions and information regarding the use of the sling and accessories. In this manual the client is the person being lifted. The caregiver is the person operating the hoist.

	This symbol indicates important information related to safety. Follow these instructions carefully.
---	--

	Read User Manual before use! It is important to fully understand the content of the user manual before attempting to use the equipment.
--	---

Visit www.etac.com for download of documentation to ensure you have the latest version.

General

	<p>CE declaration The product, and its related accessories, described in this instruction for use (IFU), is in compliance with the regulation (EU) 2017/745 of 5. April 2017 – as a medical device, risk class I. The product is manufactured and tested according to standards IEC 60601-1:2005, IEC 60601-1-2:2005 and EN/ISO 10535:2006. Any serious incident that occurs in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.</p>
---	---

Conditions for Use

Lift and transfer of a client will always pose a certain risk and only trained personnel are allowed to use the equipment and accessories covered by this user manual. The rail system must be installed by certified personnel in accordance with applicable installation instructions.

Modifications and use of components made by other manufacturers.


We recommend only using Molift components and spare parts. Declaration of conformity is not valid and Etac is not responsible for warranty if any modifications are made to the product. Etac shall not be liable for faults or accidents that can occur when using components made by other manufacturers.

	Only certified personnel are allowed to open hoist or accessories to perform service or repair. Risk of injury from rotating parts and electric shock.
---	---

The hoist is not intended to be operated by the client being lifted. If a hoist is to be used by a client living on their own, then some form of communication device shall be installed in the area of use of the hoist so that in the event of an emergency the client is able to summon assistance. This may, for example, be the fitting of an alarm system or the supply of a conveniently placed telephone, etc.

	<p>Warranty notice Two-year warranty against defects in workmanship and materials of our products. One-year warranty for batteries. Please refer to www.etac.com for terms and conditions.</p>
---	---

Molift Air Tilt

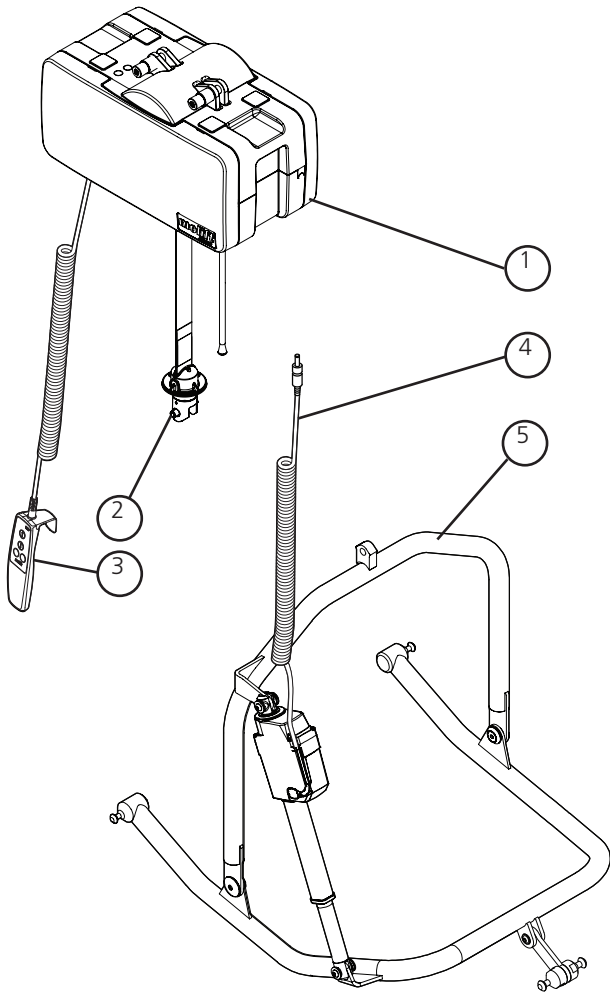Components:

1. Molift Air 350 Tilt
2. Quick release pin
3. Molift Air Tilt Hand control
4. Cable for connection to Molift Air 350 Tilt
5. Molift Air Tilt Sling bar

About Molift Air Tilt

Molift Air Tilt Sling bar is an accessory to Molift Air 350 Tilt ceiling hoist as an alternative to standard sling bars. The Molift Air Tilt has clip studs for clip sling attachment (accessory) and with the adjustable sling bar, supported by an electric actuator, it allows the caregiver to control and adjust the position of the patient, e.g. from upright to reclined.

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SN XXXXXX YYYY-MM-DD

Warning labels and Symbols

Symbols used on the product, explained in more detail:

- | | |
|--|------------------------------------|
| Compliance to all relevant EC directives | IEC 60417-5172: Class II equipment |
| Refer to user manual | Operating instructions |
| Indoor Use only | Manufacturer |
| Max user weight | Date of manufacture |
| Do not dispose in general waste
WEEE Directive 2002/96/EC | Catalogue number |
| Low battery indicator (hand control) | Serial number |
| China Pollution control mark (also indicates recyclability) China RoHS legislation | Medical Device |
| Australian Regulatory Compliance Mark – EMC, Electrical safety (RCM) | |

EN
SV
NO
DA
FI
DE
NL
FR
IT
ES

Technical data

Safe Working Load (SWL)

272 kg (600 lbs)

Weight of unit (excluding hand control and cables):

7,3 kg (16 lbs)

Actuator speed

12,6-10,4mm/second (0,49-0,40 inches/second)

Protection class:

Hand control IP24

Actuator IPX4

Operating forces button

Buttons on handset: 3.4 N

Material:

Steel, Plastic

Expected Lifetime:

Expected lifetime of 30 000 cycles SWL or 10 years.

Dimensions:



L x W x H (Length, Width)

66 x 58 x 50-79 cm

(26 x 22,8 x 19,7-33,5 inches)

Tilting Range:

75 degrees

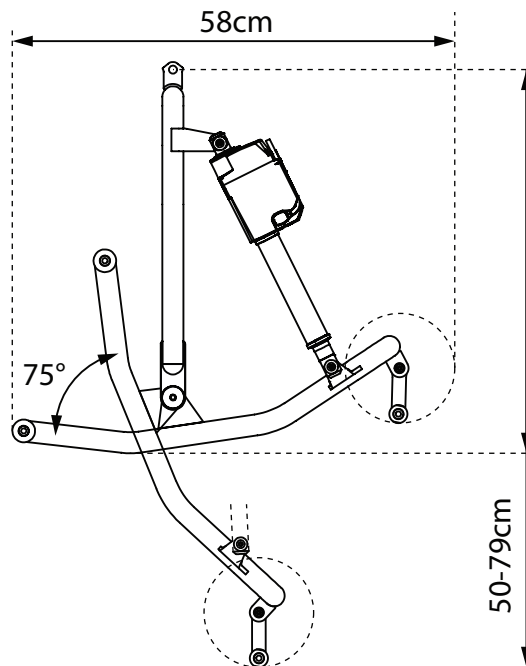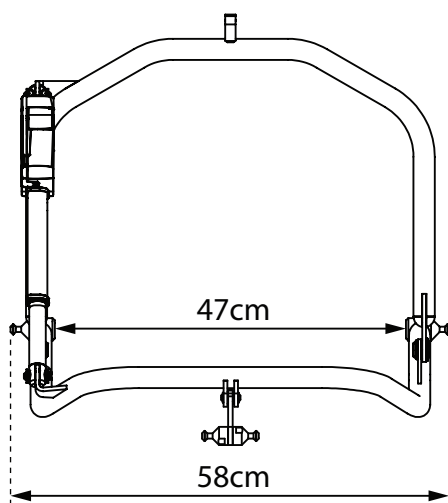Electronics

Transport and Storage

The hoist can be stored and transported under temperatures between -25 - 70 °C.

Operating

Designed for use at standard room temperatures (+5 to +40°C).

Air Pressure: 70 - 106 kPa

Relative Humidity: 15 - 93 %

After storage or transport at other temperatures leave it in a room with a suitable temperature until it reaches a safe operating temperature.

Medical electrical equipment requires special precautions regarding electromagnetic compatibility (EMC). Portable or mobile radio communication equipment may affect the medical electrical equipment, and should be kept minimum 25 cm (10 inches) from the hoists electronics.



Working pause ratio/Duty Cycle.

Duty Cycle 10%. (Intermittence according to standard ISO-EN 10535)

Installation

The hoist is marked with Safe Working Load (SWL), this should not exceed rail systems max load capacity.

Minimum required height from floor to rail is 210cm.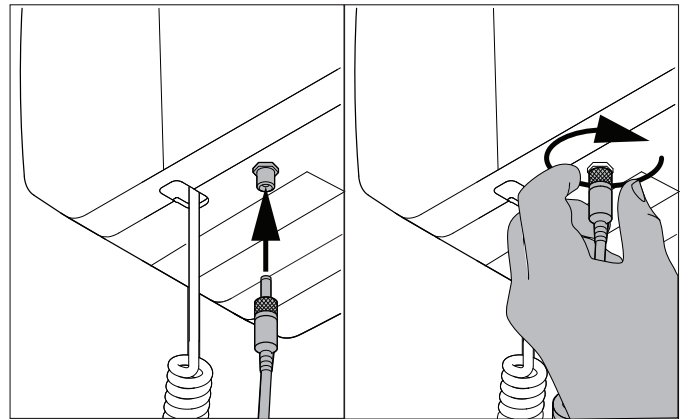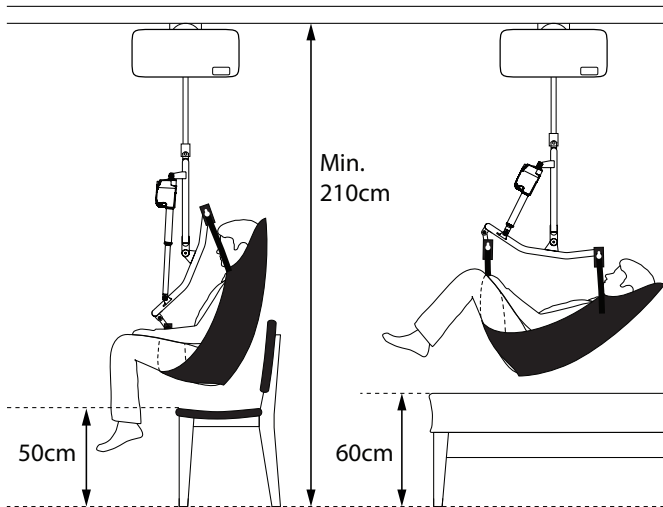


6. Insert the plug on cable from actuator on Molift Air Tilt Sling bar into the connector on Molift Air 350 Tilt.
7. Tighten the connectors sealing ring by turning it clockwise.


Hand control

The Molift Air Tilt Hand control has 2 buttons for lifting and lowering and 2 buttons for tilting the sling bar. The hand control has an indicator light that will illuminate when battery level is low and the hoist requires charging


Mounting Molift Air Tilt Sling bar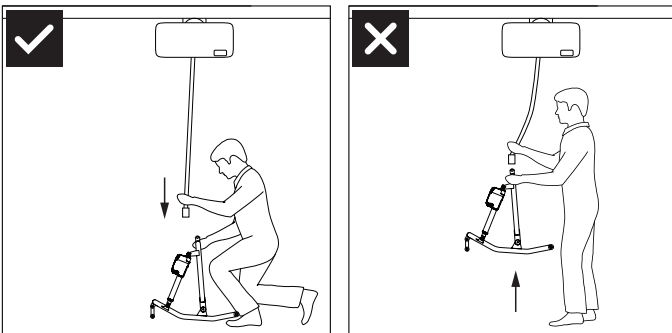

1. Place sling bar under hoist and lower lifting band. **DO NOT LIFT SLING BAR UP TO LIFTING BAND.**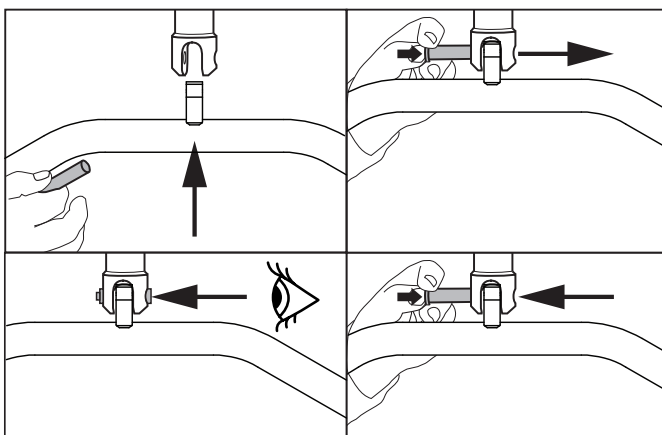

2. Align the sling bar in connection point.
3. Push button on locking pin and insert all the way through.
4. Make sure locking pin is properly fastened
5. Push and hold down the button on the locking pin, and pull to remove locking pin.

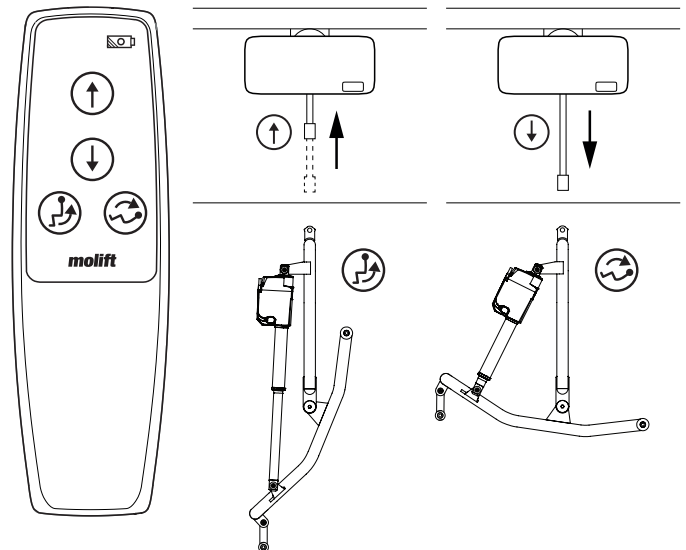Do not pull hand control to move the hoist along the rail.

Using slings

General safety precautions


Only use clip slings that are adjusted to fit the client, type of disability, size, weight and type of transfer.

 If maximum load (SWL) differs between hoist sling bar and body support unit, then the lowest maximum load shall always be used.


 Molift hoists shall only be used to lift clients. Never use the hoist to lift or move objects of any kind.

 Read User Manual for the sling prior to use.

 Do not to use damaged or badly worn slings.

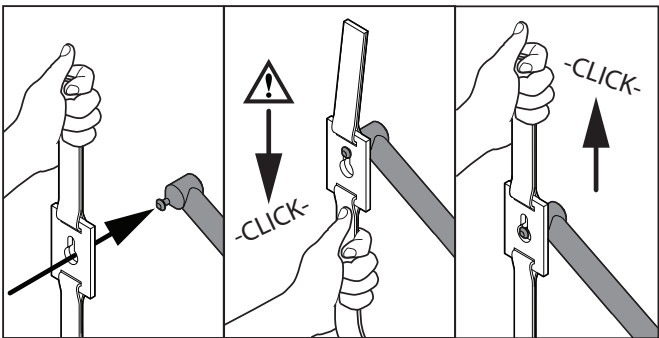
 Make sure that all 4 sling clips are correctly fitted to the clip studs on sling bar. Pull lifting band to lock clips to stud. Pull upper band to unlock.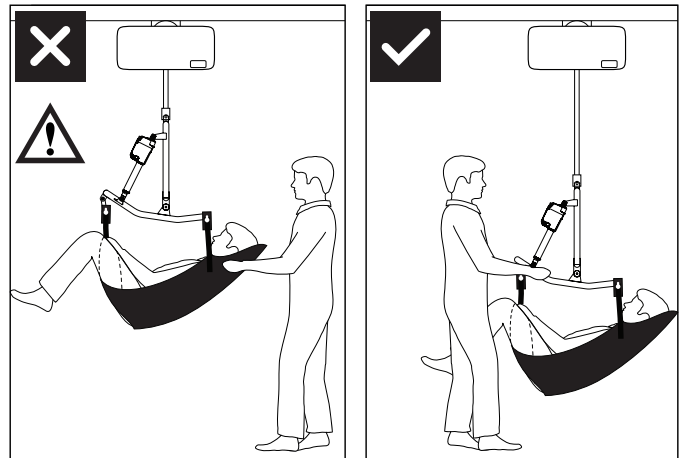



 Do not push or pull client or the sling!

Lifting and lowering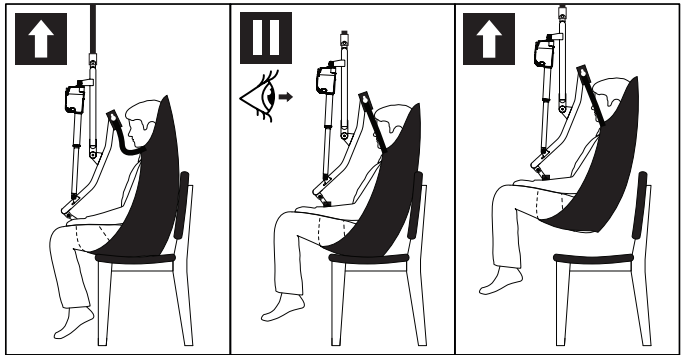

Check that the sling is correctly fitted around the client. Stretch the sling straps without lifting the client. Ensure that all four clips of the sling are securely fastened to avoid the client slipping or falling. Lift client and perform transfer.

Transfer

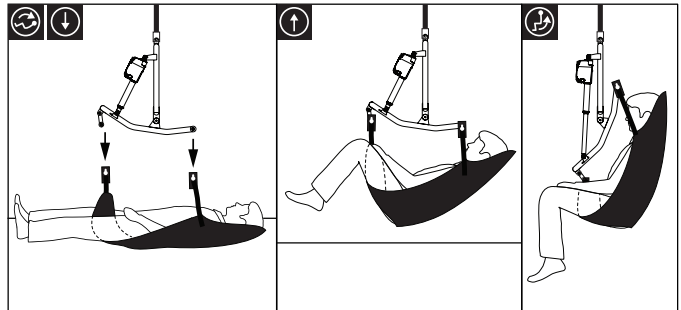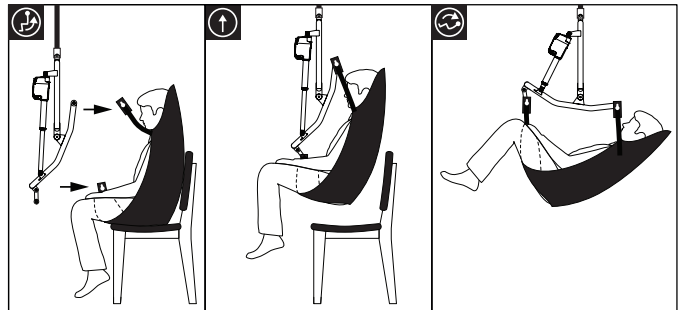Plan the lifting operation in advance to ensure that it is as safe and smooth as possible. Remember to work ergonomically. Assess the risks and take notes. The caregiver is responsible for the safety of the client. When moving the client, stand to the side of the client you are lifting. Make sure that arms and legs do not obstruct the seat, bed, etc. Keep eye contact with the client to help them feel safe.

Slings

This list of SilvaLea Clip Sling sling series are tested to be used with Molift Air Tilt Sling bar.

The SilvaLea slings are available in sizes S – XL, in polyester. Complete list of slings and sling bars can be found on etac.com.

Silvalea High Easy Toileting Sling Poly. (Toilet)

Size:	Art.no:
S	DA-ST3ASM
M	DA-ST3AMED
L	DA-ST3ALRG
XL	DA-ST3AXLRG

Silvalea High Easy Sling Poly (HighBack)

Size:	Art.no:
S	HYASMCP
M	HYAMEDCP
L	HYALRGCP
XL	HYAXLRGCP

Slings made by other manufacturers

Etac shall not be liable for faults or accidents that can occur when using slings made by other manufacturers.

Troubleshooting

Symptom	Possible Cause/Action
The hoist does not respond to hand control action.	Emergency Stop on the hoist is activated. Deactivate by pushing button back in.
	The hoists electronics is overheated. Wait for it to cool down.
	Hand control is not plugged in properly. Hand control or plug or cord can be broken and should be replaced.
	Actuator cable is not plugged in properly.

Cleaning / Disinfection

Clean on a regular basis. Clean surfaces with a damp cloth using an appropriate pH-neutral detergent. Do not use solvents or strong liquids, this may damage surfaces on the hoist. For disinfection when needed; use isopropyl alcohol. Avoid abrasive cleaning products. Do not expose electronics to running water.

Make sure not to damage or remove labels when cleaning.

Periodic inspection scope

This accessory shall be inspected as a part of the periodic inspection of the hoist it is used with.

Periodic Inspection shall be performed at least once a year or more frequently if required by local requirements. The inspection must be performed by service personnel authorized by Etac. Contact Etac at molift@etac.com for training and authorization or recommendation of an approved service partner.

If there is any problems that could jeopardize someones safety, the accessory shall immediately be taken out of service and marked "out of order". Do not use until it is repaired.

Service scope

Service involves replacing the actuator and inspection/replacement of worn parts. This must be carried out by authorized personnel. Contact your local Etac dealer, or see etac.com for a recommendation of an approved service partner.

Before use / Daily check

Inspection to be performed by caregiver daily or before use:

- Make sure sling bar connection is properly connected and secured.
- Make sure wiring, hand control, clip studs or joints do not have any visible damage.

If there are any faults or defects, the Molift Air Tilt needs to be taken out of operation and marked "out of order".

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Etac A/S