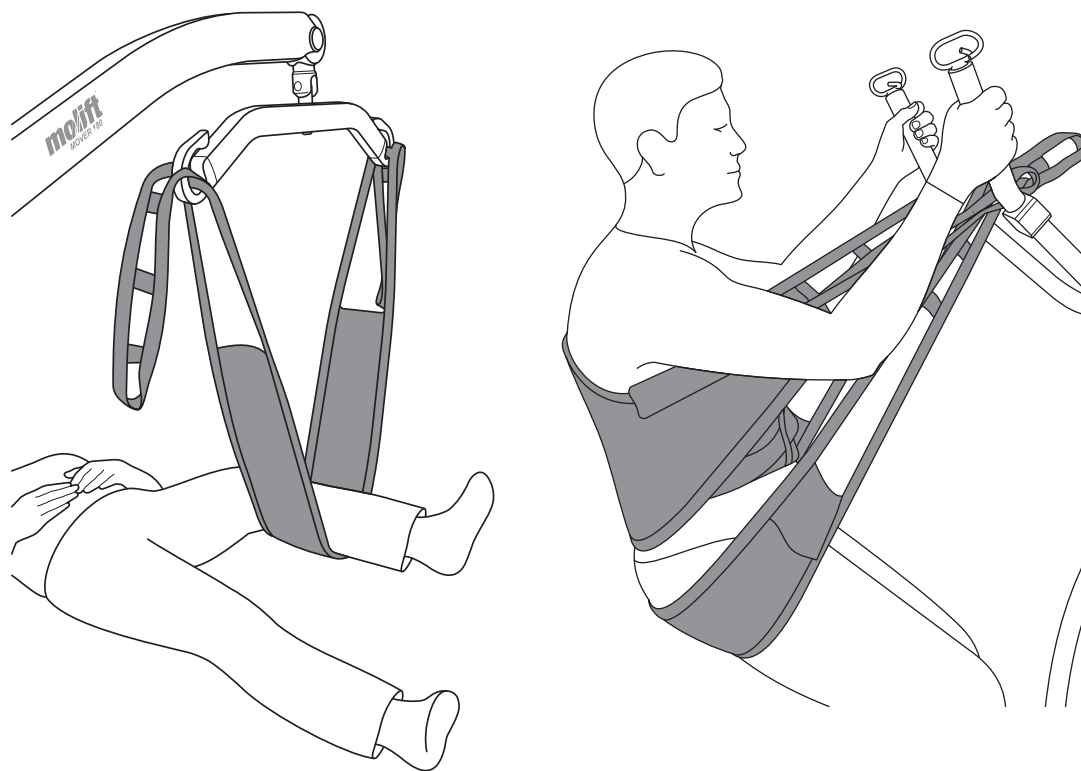
# Molift EvoSling FlexiStrap

**molift**<sup>®</sup>  
by Etac

User Manual - Bruksanvisning - Brukermanual - Brugervejledning - Käyttö-  
pas - Bedienungsanleitung - Gebruikershandleiding - Manuel d'utilisation  
- Manuale per l'utente - Manual de usuario

BM28999 Rev. B 2021-02-15

MD

CE

## Contents

|                                 |      |
|---------------------------------|------|
| User manual - English           | - 4  |
| Bruksanvisning - Svenska        | - 8  |
| Brukermanual - Norsk            | - 12 |
| Brugsvejledning - Dansk         | - 16 |
| Käyttöohje - Suomi              | - 20 |
| Gebrauchsanweisung - Deutsch    | - 24 |
| Handleiding - Nederlands        | - 28 |
| Manual d'utilisation - Français | - 32 |
| Manual utente - Italiano        | - 36 |
| Manual de usuario - Español     | - 40 |

## General

### Declaration of conformity

The Molift EvoSling FlexiStrap and related accessories described in this operator manual are CE marked in accordance with EU Council Directive MDR (EU) 2017/745 concerning medical devices, class 1, and has been tested and approved by a third party according to standard EN ISO 10535:2006.

**Conditions for Use:** Lift and transfer of a person will always pose a certain risk and only trained personnel are allowed to use the equipment and accessories covered by this user manual.

**Warranty:** 2-year warranty against defects in workmanship and materials of our products. For Terms and conditions, see [www.etac.com](http://www.etac.com)

Visit [www.etac.com](http://www.etac.com) for download of documentation to ensure you have the latest version.

## Important

This User Manual contains important safety instructions and information regarding the use of the sling and accessories. In this manual the user is the person being lifted. The assistant is the person operating the lifter.

This symbol indicates important information related to safety. Follow these instructions carefully.

**Read User Manual before use!**  
It is important to fully understand the content of the user manual before attempting to use the equipment. Read the manual for both lift and sling

**General Safety Precautions:** Molift FlexiStrap is a lifting band designed for different uses such as lifting up and holding a body part and providing support to the user. Only use accessories and slings that are adjusted to fit the user, type of disability, size, weight and type of transfer. It is important that the sling has been tested with the individual user and for the intended lifting situation. Make a decision on whether one or more assistants are required. Plan the lifting operation in advance to ensure that it is as safe and smooth as possible. Remember to work ergonomically.

Assess the risk and take notes.

You as the assistant are responsible for the safety of the user!

Molift slings shall only be used to lift persons. Never use the sling to lift or move objects of any kind.

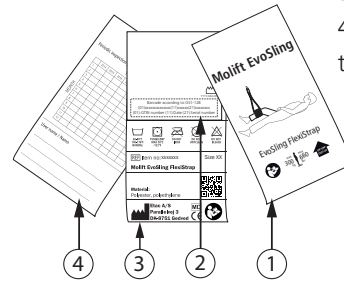
If maximum load (SWL) differs between lifter, suspension and body support unit, then the lowest maximum load shall always be used

The sling should not be stored in direct sunlight.

## Labels and symbols

1. Main label
2. Batch and serial number label w. GS1-128 barcode.
3. Product label
4. Periodic inspection label/ Name tag

### Symbols:

Manufacturer

Date of manufacture. YYYY-MM-DD (year/month/date)

CE marked

Refer to user manual

This way up, this side is outside

Max user weight

Medical Device

### Wash symbols:

Slings can be washed at temperatures ranging from 60 up to 85° C

Tumble drying, max 60° C.

Do not iron

Do not dry clean

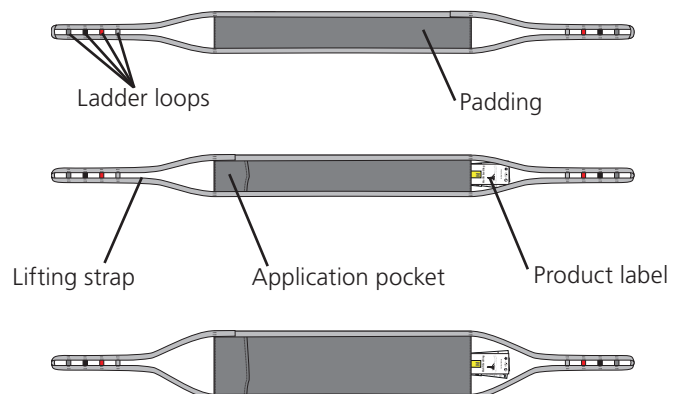
Do not bleach

## About EvoSling FlexiStrap


Molift FlexiStrap is a lifting band designed for different uses such as lifting up and holding a body part and providing support to the user. It may be used in situations such as treatment or examination of ulcers or foot care. FlexiStrap, standard model, can be used as an accessory to Nova Standup sling for users who need extra support. Molift EvoSling FlexiStrap is available in two sizes, Standard and Wide, in padded polyester.

The sling has an application pocket to facilitate application of sling. Choose the size that best suits the user's needs and the activity to be performed. Keep in mind that the patient may be sensitive to pressure, touch and pain in the area where the hoisting belt should be placed. Make sure that the user has the mobility of the joints required to lift the body part.

**Technical data**

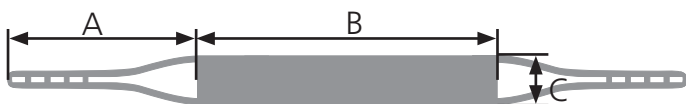Safe Working Load (SWL):      Material:  
 XXS-XL: 300 kg (660 lbs)      Polyester, Polyethylene

| Size     | A  | B  | C  |
|----------|----|----|----|
| Standard | 60 | 95 | 15 |
| Wide     | 60 | 95 | 25 |


**Expected Lifetime:**

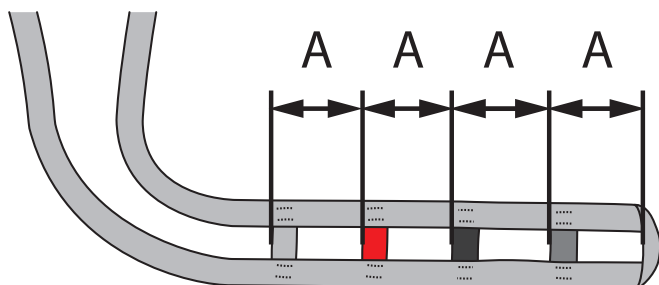The product has an expected service life of 1 to 5 years under normal use. The service life of the product varies depending on usage frequency, materials, loads and how often it is washed. Washing at higher temperatures wears out the material faster.


**Lifting Strap Ladder loops**

The ladder loops on the lifting strap give different options for mounting the sling to suspension. Use the ladder loop that makes sling most comfortable for the user.

The ladder loops on the straps have different colours so the strap can easily be fastened with equal length on the left and right side. The distance between each ladder loop (A) is 9cm.

**Combination list**
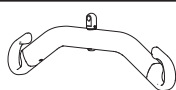
| <br>2 Point Suspension | Sling size |     |        |
|---|------------|-----|--------|
|   | XS-S       | M-L | XL-XXL |
| Small 340mm   | ✓          | ✓   | ✓      |
| Medium 440mm  | ✓          | ✓   | ✓      |
| Large 540mm   | ✓          | ✓   | ✓      |

**Before use / Daily check**

 Before using sling for first time it should be marked with the date of first time use on periodic inspection label.

Inspection to be performed daily or before use:

- Make sure user has the required ability for the sling
- Make sure the sling suitable for the lift/ suspension to be used.
- Make sure sling does not have visible damage or frays.

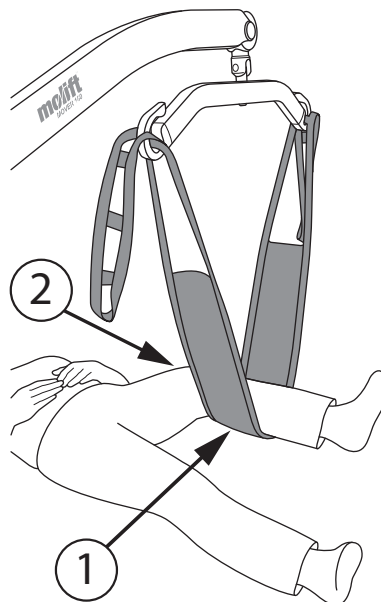
**Periodic inspection**

Periodic inspection of the non-rigid body support unit should be undertaken at the time intervals stated by the manufacturer, but at least every 6 months. More frequent inspections may be required where a non-rigid body support unit is used or cleaned more frequently than normal

 Never use a faulty or damaged sling as it can break and cause personal injury. Destroy and discard damaged and old slings


 Make sure not to damage or remove labels when cleaning.

**Sling checkpoints**


1. The sling is placed under user as required with no folds
2. Do not lift higher than the users mobility of the joints can sustain.


## Sling application

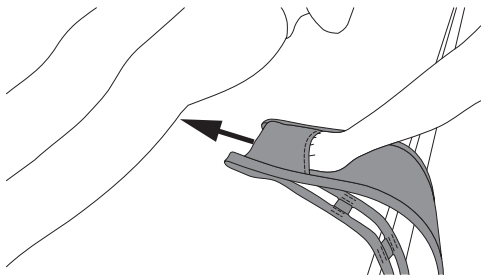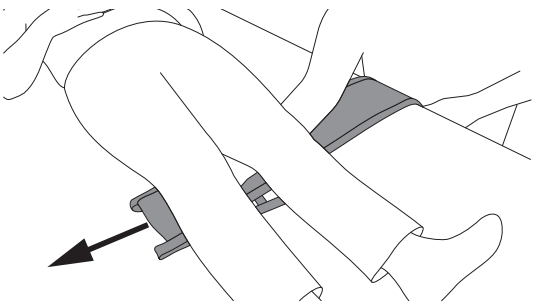
1. Use the application pocket on the sling for easier fitting.

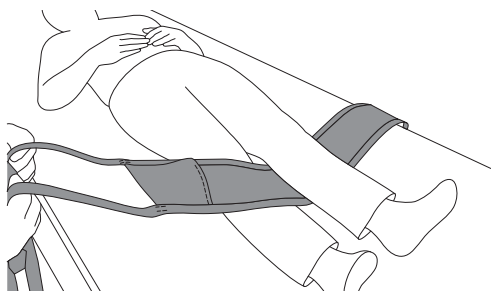2. Push the sling under the user so it is possible to pull out on the opposite side.

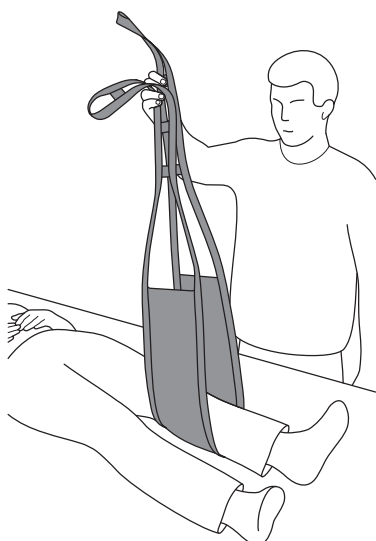
3. Pull the sling and center it under users body part(s)

4. Make sure the lifting straps are fastened with equal length on right and left side.

## Alternative use

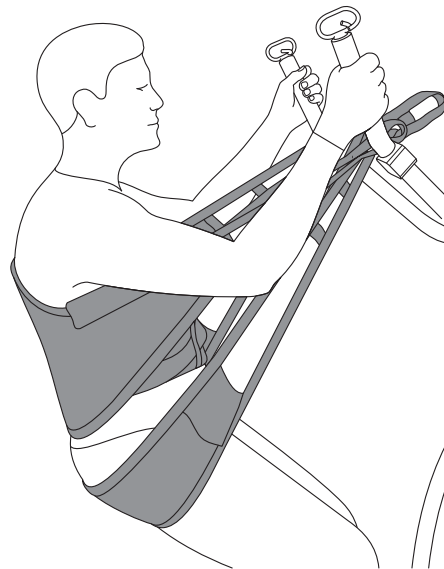
EvoSling FlexiStrap Standard can be used as an accessory for the Etac Nova Stand Up.

Please read the user manual for Etac Nova Stand Up Sling and Nova 500NG before use! It is important to fully understand the contents of the user manual before attempting to use the equipment.

## Transfer

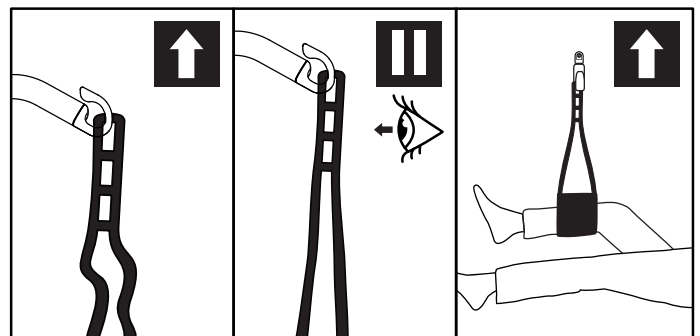When repositioning the user, stand to their side. Make sure that arms and legs do not obstruct the seat, bed, etc. Make sure the user has the mobility necessary for the intended hoisting procedure.

Never lift the users body part higher than necessary. Remember that wheels on a mobile lift must NOT be locked.

1. Check that the sling is correctly fitted around the user and that the strap loops are correctly fitted to the suspension hooks.
2. Start lifting until the lifting straps are stretched without lifting the users body part. Ensure that both loops of the sling are securely fastened.
3. Lift the users body part.

This sling cannot be used for point to point transfer! It is only for lifting body parts or repositioning of user.

Never leave a user unattended in a lifting situation.

The hoist should not be used to lift or move users on sloping surfaces

Remove the sling gently to avoid injuring the user.


### Recycling

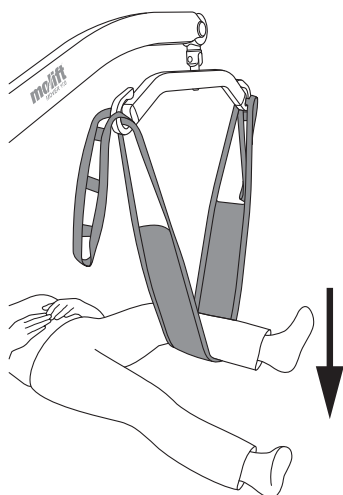Recycling instructions is available at [Etcac.com](http://Etcac.com)


### Troubleshooting

| Symptom   | Possible Cause/Action           |
|---|---------------------------------|
| The sling is in the way of the area that needs treatment.   | Try the standard sized version. |
| The sling applies too much pressure on the users body part. | Try the wide sized version.     |

### Removing the sling

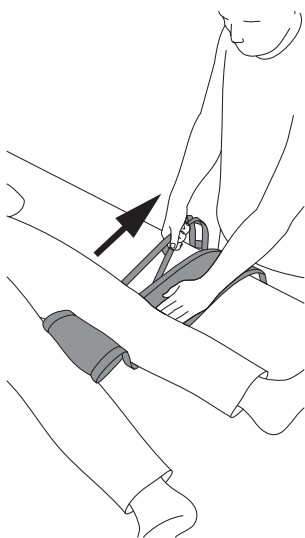1. Lower the users body part. Release the lifting band from the suspension.

2. Remove the sling by folding the lifting band under the sling itself and and pull out

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Etac A/S