ART. 4CN.445.xxxx.xx (C)

Users manual Rev.3 2021-07

SOSTEGNO DI SICUREZZA RIBALTABILE ESTRAIBILE / EXTRACTABLE LIFT-UP SAFETY SUPPORT / BARRE DE SÉCURITÉ RABATTABLE EXTRACTIF / STÜTZKLAPPGRIF, ABNEHMBAR

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EN INSTRUCTIONS

Read before installation, use and maintenance. This document is an integral part of the product: please keep it. All sizes are in mm. If you cannot read this document, it is possible to download this documentation on our website www.pba.it.

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EN
This article complies with the MDR regulation (2017/745/EU) concerning medical devices.

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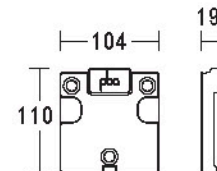
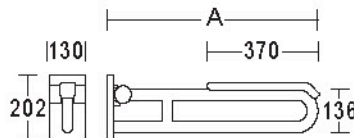
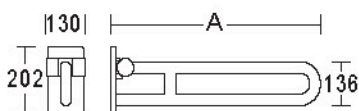
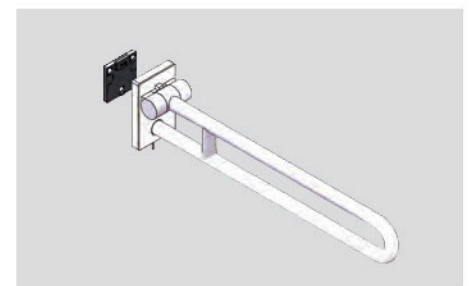
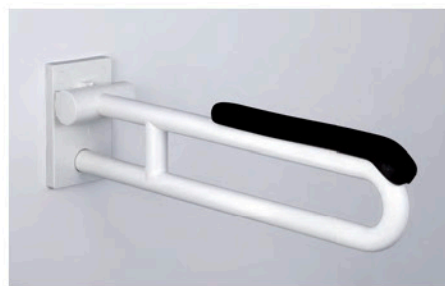
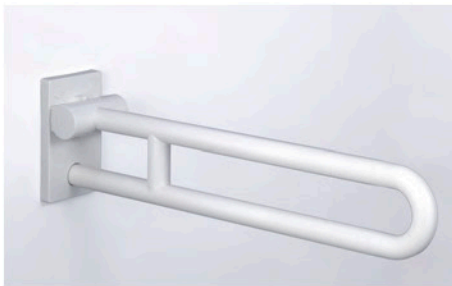
EN INTENDED USE
The lift-up supports are aids for disable or aged people. The lift-up safety supports are wall-mounted, especially in bathrooms, on both sides of the sink and the toilet. The lift-up safety supports offer a rest and support aid and are used by people with physical impairments to help them stand up safely from the seated position, with or without the aid of an assistant.

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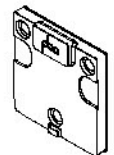
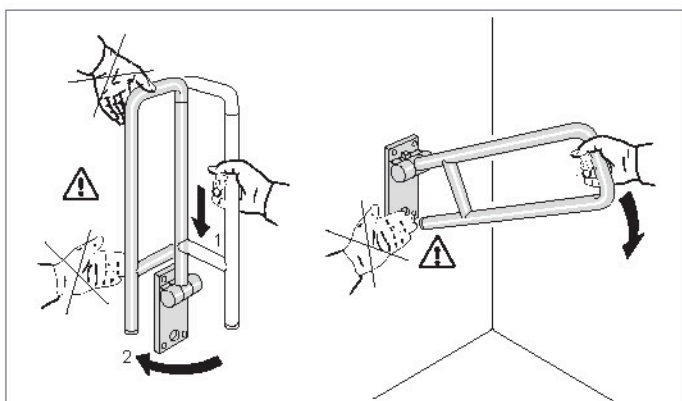
EN - APPLICATIONS AREA

Hospitals, rehabilitation centres, nursing home for elderly, public buildings, private bathrooms.

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EN SAFETY PRECAUTIONS AND INDICATIONS FOR MAINTENANCE

WARNING: All disabled accessories must be fixed in accordance with the table of fixing materials. pba S.p.A. is not liable for any damage/loss of functionality of the product caused by not observing the recommendations set out in our instruction manual. Under NO circumstances must the pba S.p.A. disabled and bathroom accessories be altered, tampered with or modified. Non-observance of these instructions makes product guarantees null and void.

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EN

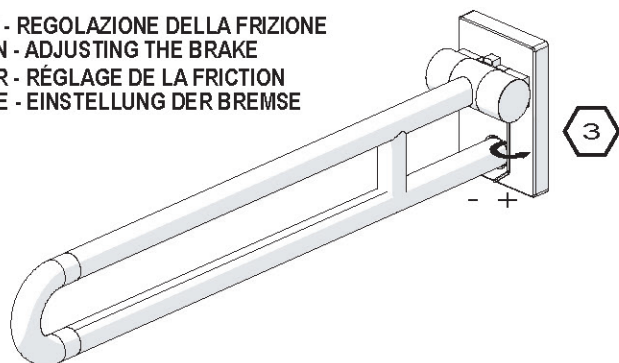
SAFETY PRECAUTIONS AND INDICATIONS FOR MAINTENANCE

Do not use the lift-up supports improperly or for purposes other than that indicated in the instruction manual. Avoid installing the supports inside saunas or inside too much hot rooms (a temperature of the support's surface exceeding 41°C could cause skin damages). If the support doesn't work properly, contact pba technical office (TEL +039 0424 5451) or pba Deutschland GmbH (+049 2841 99890 0).

WARNING: Before using, make sure that the removable lift-up safety support is fixed to the 4CN.445.0CPM.00 wall plate with the appropriate grub screw following our below mentioned instructions. Once the grub screw is screwed it must be flush with the plastic plate.

On a periodic basis (at least once a month), make sure the support lowers slowly and smoothly; if necessary, to increase the braking effect of the clutch, simply adjust the actual clutch following the instructions written below. Also make sure, on a periodic basis, that the wall-mounting bracket is not damaged or has not worked loose. The application of a load heavier than the maximum weight indicated could permanently damage the lift-up safety support and its wall-mounting bracket. If this should occur, and also if a permanent deformation of the support is visible, contact pba SpA (TEL +039 0424 5451) or pba Deutschland GmbH (+049 2841 99890 0). Before lowering the support, make sure nobody is near where it moves. The failed observance of these instructions could cause serious damages and injuries.

IT - REGOLAZIONE DELLA FRIZIONE
EN - ADJUSTING THE BRAKE
FR - RÉGLAGE DE LA FRICTION
DE - EINSTELLUNG DER BREMSE

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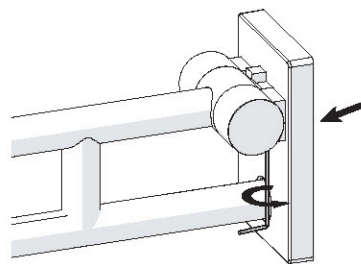
EN
INSTALLATION PROCEDURES BY STANDARD FIXING EQUIPMENT

The following installation procedures refer to the fixing of disabled hardware to concrete. For other types of building construction, refer to the table of fixing material.

- ⚠ Before installing the item, write down the serial number on the provided form. The installation must be carried out by competent personnel.

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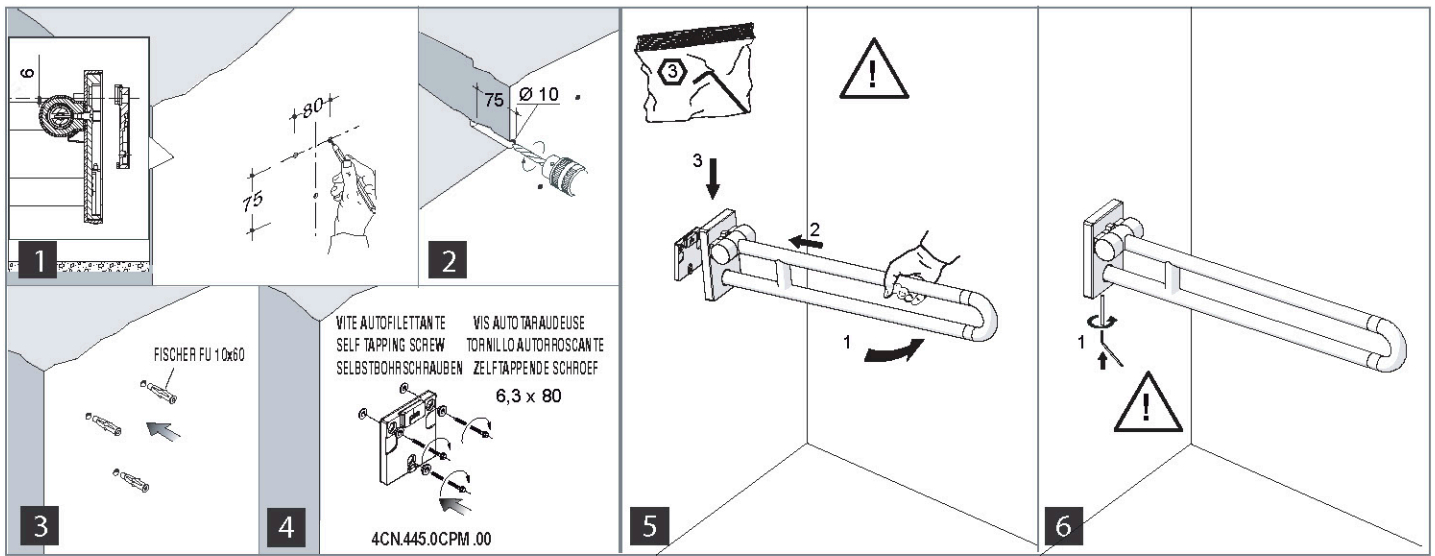
EN
NOTE: for fixing to other materials contact the pba technical office.

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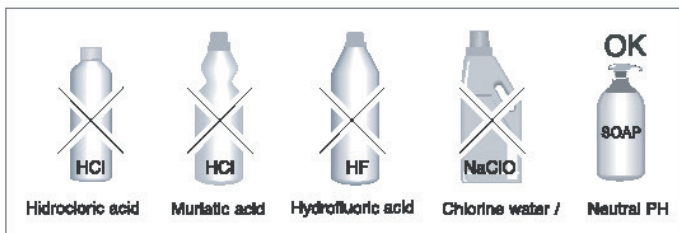
**EN
INDICATIONS FOR CLEANING**

The products only require simple cleaning and maintenance.
To clean pba products periodically use a wet cloth. As an alternative use neutral detergents or disinfectants diluted in a solution having neutral PH. Es. Microbak Forte (Praxidienst). After washing rinse the products with cool water and let them dry. Always select the correct soaps/disinfectants in accordance with the manufacturer's instructions (ensuring that they are compatible with the different surfaces and correct concentration). It is recommended to test the detergent/disinfectant on a small hidden area of the product before use. The use of detergents containing chlorine, acids and alkali, organic solvents and abrasive substances is not recommended. Do not use any abrasive objects (like brushes) for the cleaning. Pba S.p.A. declines all responsibility for damages caused by use of unsuitable cleaning methods listed above.

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**EN
INFORMATION FOR THE DISPOSAL OF WASTE PRODUCTS**

At the end of its life, the product must be disposed according to the local waste regulation. The article is made of recyclable polyamide and all the metallic components are recyclable.

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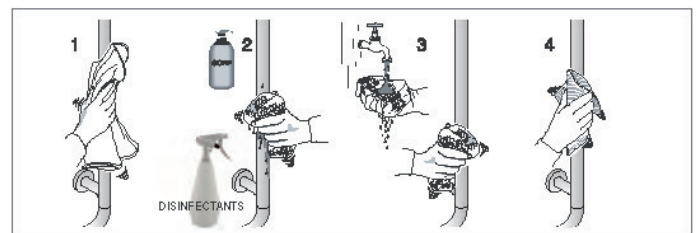
EN
Under the intended conditions of use and in accordance with the guidelines for maintenance and cleaning, the expected life time of the product is of 5 years.

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WALL RESISTENCE

The wall must have sufficient strength to ensure the application of 250 kg at each fixing point.

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MAINTENANCE

- 1) Check that the support is well anchored to the wall.
- 2) If the lift-up support does not remain in a vertical position, tighten the specific grub screw positioned under the shell.
- 3) In case of breakage of any component or in case of anomalies, contact immediately the manufacturer and do not use the support.
- 4) It is recommended to carry out maintenance checks every 1 year. Record the checks carried out in the appropriate attached form.

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