ART. 4CN.447.xxxx.xx
MPC.IMB.ISTRUZ.113

Users manual Rev.5 2021-07

SEGGIOLINO RIBALTABILE / LIFT-UP SHOWER SEAT / SIÈGE RELEVABLE / DUSCHSITZ KLAPPBAR

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EN INSTRUCTIONS

Read before installation, use and maintenance. This document is an integral part of the product: please keep it. All sizes are in mm. If you cannot read this document, it is possible to download this documentation on our website www.pba.it.

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EN
This article complies with the MDR regulation (2017/745/EU) concerning medical devices.

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EN INTENDED USE
Lift-up shower seats are a valid aid for elderly or disabled people. They are fixed to walls in shower areas and offer the possibility for people, with or without physical handicap to sit whilst showering.

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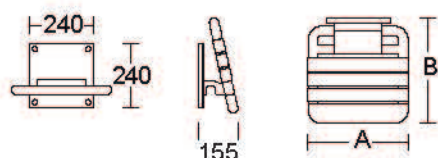
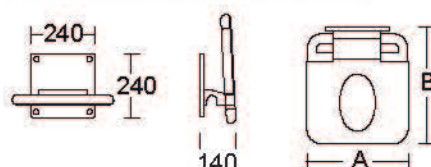
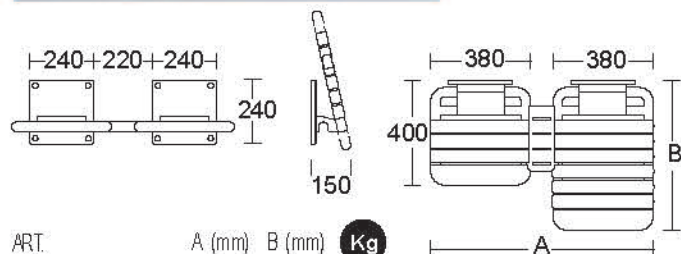
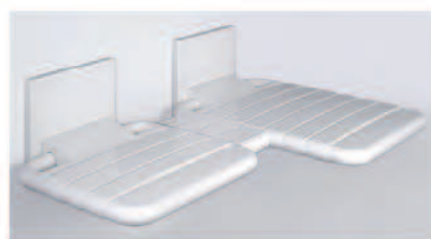
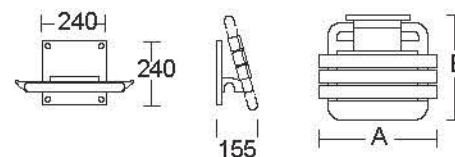
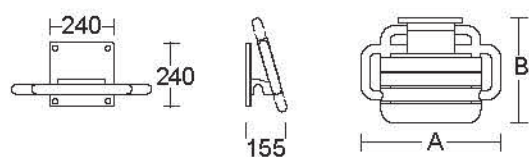
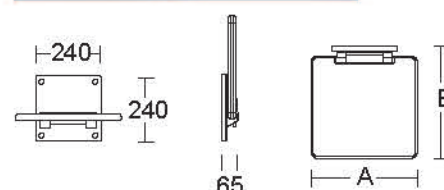
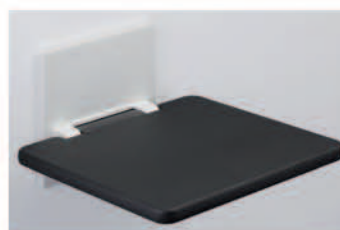
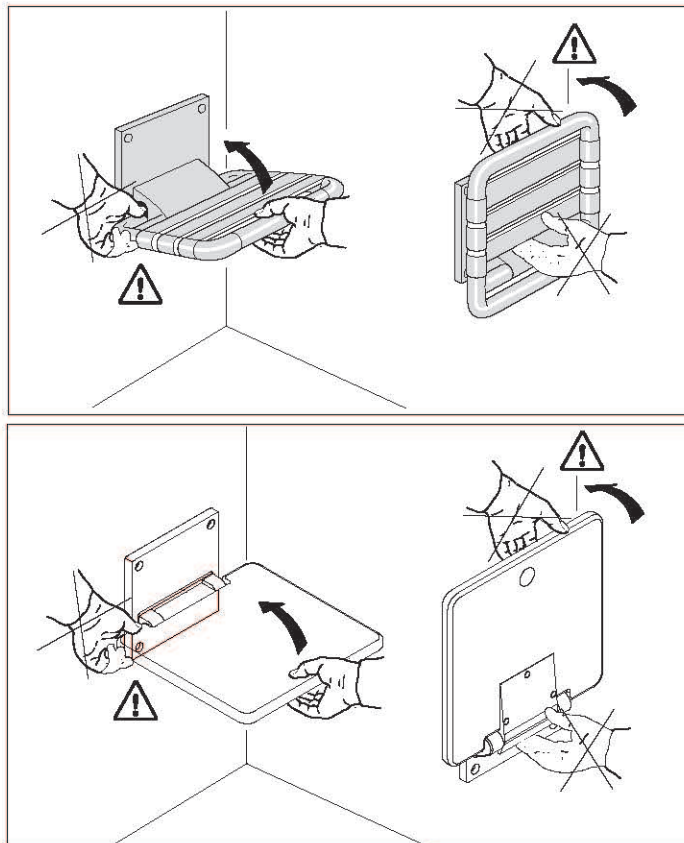
EN - APPLICATIONS AREA

Hospitals, rehabilitation centres, nursing home for elderly, public buildings, private bathrooms.

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EN

Do not use the shower seats improperly or for any different use other than those indicated in the instruction manual. Shower seats must not be installed inside saunas or in extremely hot areas (a temperature of the support's surface exceeding 41°C could cause skin damages). In case of malfunctioning of our showerseats, please contact the technical department at pba S.p.A. (Tel. +39 (0)424 5451) or pba Deutschland GmbH (+49 2841 998900). It is recommended to check the correct functioning of the showerseat periodically (once a month) and that the fixation at wall of the showerseat is not damaged or loose. Overloading the seat may cause permanent damages to the shower seat and to the fixing system. In this case, please contact pba S.p.A. (Tel. +39 (0)424 5451) or pba Deutschland GmbH (+49 2841 998900). On a periodic basis (at least once a month), make sure the shower seat lowers slowly and smoothly. Failure to regard these precautions may cause serious damage or accidents.

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EN

SAFETY PRECAUTIONS AND INDICATIONS FOR MAINTENANCE

WARNING: All disabled accessories must be fixed in accordance with the table of fixing materials. pba S.p.A. is not liable for any damage/loss of functionality of the product caused by not observing the recommendations set out in our instruction manual. Under NO circumstances must the pba S.p.A. disabled and bathroom accessories be altered, tampered with or modified. Non-observance of these instructions makes product guarantees null and void.

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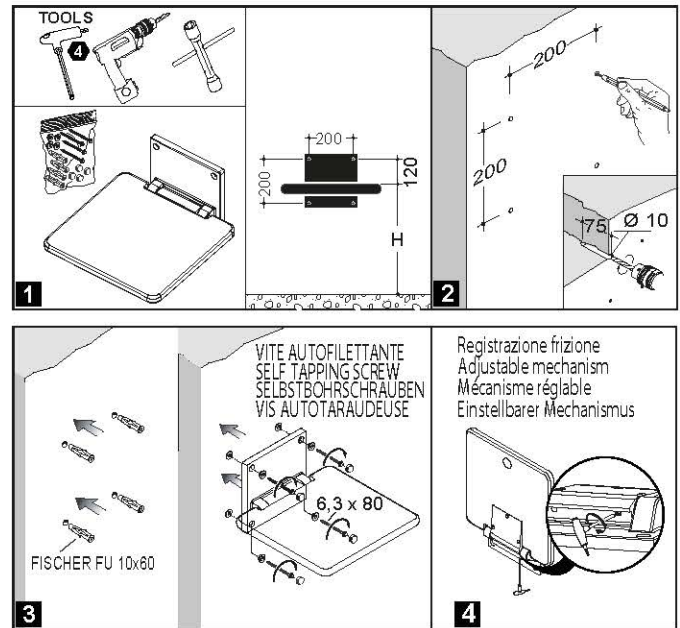
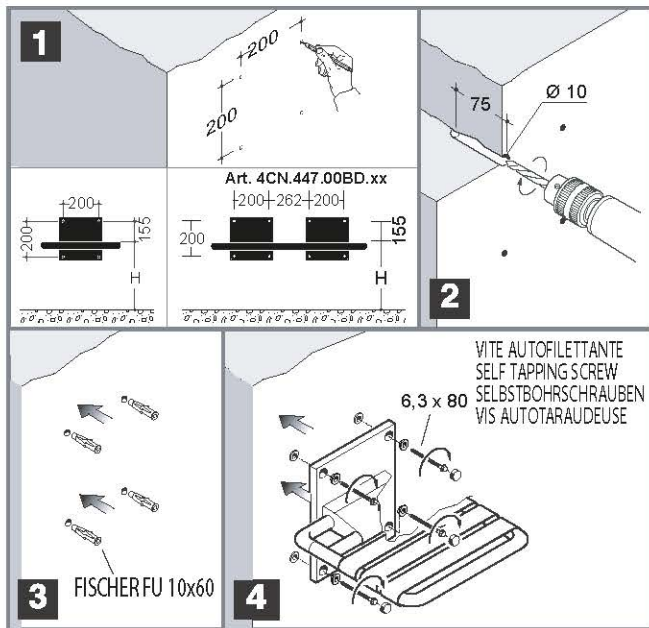
EN
INSTALLATION PROCEDURES BY STANDARD FIXING EQUIPMENT
 The following installation procedures refer to the fixing of disabled hardware to concrete. For other types of building construction, refer to the table of fixing material.
 ⚠ Before installing the item, write down the serial number on the provided form. The installation must be carried out by competent personnel.

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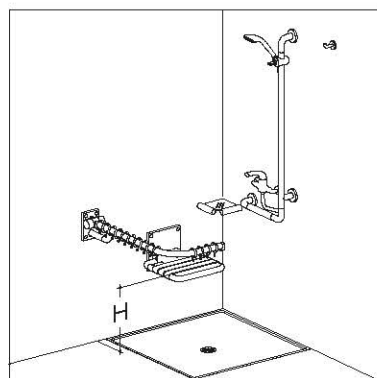
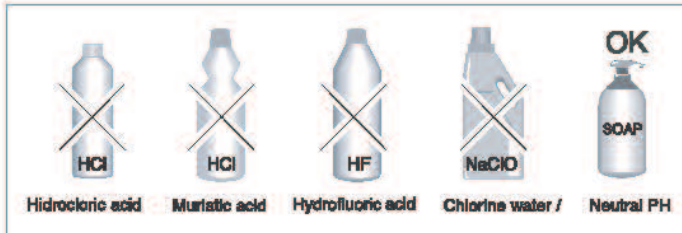
**EN
INDICATIONS FOR CLEANING**

The products only require simple cleaning and maintenance. To clean pba products periodically use a wet cloth. As an alternative use neutral detergents or disinfectants diluted in a solution having neutral PH. Es. Microbak Forte (Praxidienst). After washing rinse the products with cool water and let them dry. Always select the correct soaps/disinfectants in accordance with the manufacturer's instructions (ensuring that they are compatible with the different surfaces and correct concentration). It is recommended to test the detergent/disinfectant on a small hidden area of the product before use. The use of detergents containing chlorine, acids and alkali, organic solvents and abrasive substances is not recommended. Do not use any abrasive objects (like brushes) for the cleaning. Pba S.p.A. declines all responsibility for damages caused by use of unsuitable cleaning methods listed above.

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**EN
INFORMATION FOR THE DISPOSAL OF WASTE PRODUCTS**

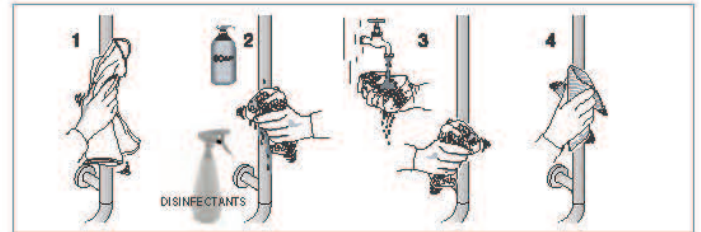
At the end of its life, the product must be disposed according the local waste regulation. The article is made of recyclable polyamide and all the metallic components are recyclable.

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EN

Under the intended conditions of use and in accordance with the guidelines for maintenance and cleaning, the expected life time of the product is of 5 years.

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EN
NOTE: for fixing to other materials contact the pba technical office.

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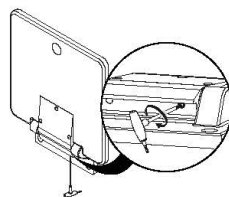
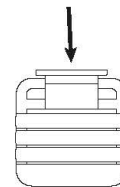
WALL RESISTENCE

The wall must have sufficient strength to ensure the application of 250 kg at each fixing point.

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MAINTENANCE

- 1) Check that the support is well anchored to the wall.
- 2) Verify the mobility and the correct folding of the seat.
- 3) In case of breakage of any component or in case of anomalies, contact immediately the manufacturer and do not use the support.
- 4) It is recommended to carry out maintenance checks every 1 year. Record the checks carried out in the appropriate attached form.

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PERIODIC INSPECTION LOGBOOK

Date	Serial number APPH	Informations about conditions	Corrective actions	Next inspection Date	Signature

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